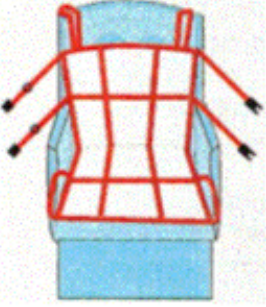
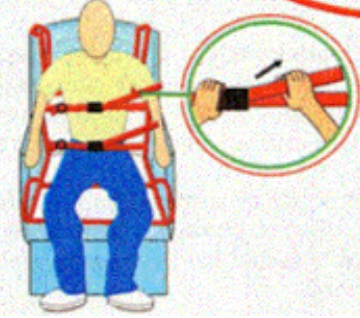
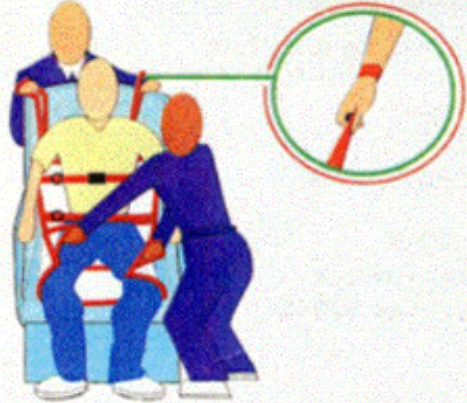
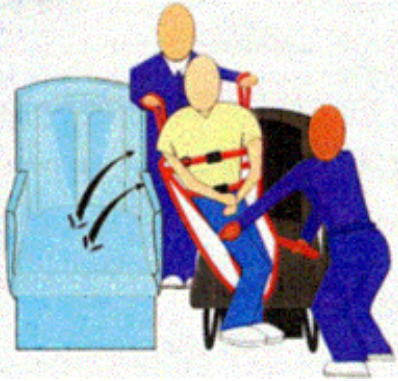


Personal Mobility Sling Rescue and Evacuation Device

Made in the USA

Patent # 6,276,006

<p>Step One</p>  <p>Position Pakkie sling on seat as follows:</p> <ul style="list-style-type: none"> • White side facing up with web straps showing. • Top loop handles over the top of seat back. • Bottom loop handles at bottom edge of seat cushion. 	<p>Step Two</p>  <ul style="list-style-type: none"> • Buckle upper security strap under person's arms and adjust snug across chest. • Buckle lower security strap and adjust snug across lap. • Adjust to fit using loose straps. (Shown in inset) 	
<p>Step Three</p>  <ul style="list-style-type: none"> • One staff member stands behind seat, puts hands through Pakkie's top loops and grasps straps. (Shown in inset) • Second staff member stands next to the seat front and grasps bottom loop handles. 	<p>Step Four</p>  <ul style="list-style-type: none"> • At the count of three, staff members lift person up from seat, over arm rests and into second seat. 	<p><u>CAUTION</u> READ BEFORE USING</p> <ul style="list-style-type: none"> • Inspect straps and hardware before each use. • Do not use if worn or torn edges are present. • Larger persons may require more than two staff members to lift. <p><u>CARE AND MAINTENANCE</u></p> <ul style="list-style-type: none"> • Do not bleach • Machine wash cold • Mild detergent • Tumble dry-low setting • Do not iron

For more information please contact Robert Watson at robert@pakkie.com or 301-343-7251